



### Instructions for Use

**Product : Suction / Evacuation Tip**

- 1) Saliva Ejector – SE 001 (Clear), SE 002 (Color)
- 2) High Volume Suction Tube – HVST 001 (Color), HVST 002 (White/Clear)
- 3) Vented Oral Evacuation Tip – VE 001 (White), VE 002 (Color)
- 4) Surgical Tip – ST 001 White – 1/8", ST 002 Green – 1/4", ST 003 Blue – 1/16"

**Intend Use:** The device is used to suction saliva and debris from the patient's mouth during dental surgical procedures.



**Cautions:**

1. Please check if the packaging is completed before use.
2. It's for single use only.

**Instruction for use:**

- 1 Connect the tube with the dental unit
- 2 Put the tip inside the mouth
- 3 Turn on dental unit for suction / evacuation
- 4 Dispose after single use.

**Storage:** The product should be stored in dry place and keep away from acid, heat and direct sunlight during storage and transportation.

**Shelf life:** 5 years

**Labels, Packing Logo Design:**

Symbol	Introductions	Symbol	Introductions
	Lot Number / Batch Code		Do not reuse" are "single use, "Use only once
	Warnings and Precautions		non-sterile
	medical device		Manufacture Date





# Sif Medical Manufacturing

	Manufacturer Name & Address		Expiry Date
	No direct sunlight		Keep dry

## Manufacturer Information



Saudi Industrialists Factory for Plastic Products  
Factory No.105, Zone A,  
Sudair City for Industry & Business, Riyadh - KSA  
Tel: +966 112774111 / +966 553469922  
Website: [www.sif-manufacturing.com](http://www.sif-manufacturing.com) E-mail: [info@sif-manufacturing.com](mailto:info@sif-manufacturing.com)

**Note :** Occurrence of any serious incidents should report to the manufacturer and SFDA (19999) immediately.



 00966553469922

 [info@sif-manufacturing.com](mailto:info@sif-manufacturing.com)

 BOX 88992-RIYADH 11692

 [sif-manufacturing.com](http://sif-manufacturing.com)