



**Sif Medical**  
Manufacturing

## Instructions for Use

**Name:** Cotton Roll

**Intend Use:** The Cotton Roll is intended to be used in dental for absorbing saliva and other fluids to help keep the area free of moisture.



### Cautions:

1. Please check if the packaging is completed before use.
2. It's for single use only.

### Cotton Roll - Instruction for use:

- **Preparation:** Wash and dry your hands thoroughly before handling the cotton rolls.
- **Placement:** Gently place the cotton roll(s) in the patient's mouth at the beginning of the dental procedure. Position them in a manner that provides optimal absorption and comfort without obstructing the treatment area.
- **During Treatment:** Monitor the cotton rolls and replace them as needed to maintain a dry and clean working environment.
- **Removal:** Carefully remove the cotton rolls from the patient's mouth after the procedure is completed. Ensure that no cotton fibers are left behind.
- **Disposal:** Dispose of used cotton rolls in accordance with your dental practice's waste disposal protocols. Do not reuse cotton roll to prevent contamination.

**Storage:** Store in the well ventilated rooms, temperature control  $\leq 40^{\circ}\text{C}$ , moisture  $\leq 80^{\circ}\text{C}$ . Keep away from acid and avoid direct sunlight during storage and transportation.

**Shelf life:** 5 years



 00966553469922

 [info@sif-manufacturing.com](mailto:info@sif-manufacturing.com)

 BOX 88992-RIYADH 11692

 [sif-manufacturing.com](http://sif-manufacturing.com)



# Sif Medical Manufacturing

## Labels, Packing Logo Design:

Symbol	Introductions	Symbol	Introductions
	Lot Number / Batch Code		Do not reuse" are "single use, "Use only once
	Warnings and Precautions		non-sterile
	medical device		Manufacture Date
	Manufacturer Name & Address		Expiry Date
	Do not use if package is damage		Consult Instructions for use
	No direct sunlight		Keep dry
	Upper limit of temperature		Upper limit of humidity



00966553469922

info@sif-manufacturing.com

BOX 88992-RIYADH 11692

sif-manufacturing.com



# Sif Medical Manufacturing

## Manufacturer Information



Saudi Industrialists Factory for Plastic Products  
Factory No.105, Zone A,  
Sudair City for Industry & Business, Riyadh - KSA  
Tel: +966 112774111 / +966 553469922  
Website: [www.sif-manufacturing.com](http://www.sif-manufacturing.com) E-mail: [info@sif-manufacturing.com](mailto:info@sif-manufacturing.com)

## Difference between models :

The main difference between the below three models are its size.

## Cotton Roll Models :

Cotton Roll CR 001 – 1-1/2" x 5/16"

Cotton Roll CR 002 – 1-1/2" x 3/8"

Cotton Roll CR 003 – 1-1/2" x 1/2"

The difference between the above three models are its diameter while keeping the length same as 1-1/2" and diameter follows as  $\varnothing$  5-16", 3/8" and 1/2 ".

**Note :** Occurrence of any serious incidents should report to the manufacturer and SFDA (19999) immediately.



00966553469922

[info@sif-manufacturing.com](mailto:info@sif-manufacturing.com)

BOX 88992-RIYADH 11692

[sif-manufacturing.com](http://sif-manufacturing.com)