



Sif Medical
Manufacturing

Instructions for Use

Name: Cotton Pellets

Intend Use: The Cotton Pellets is intended to be used in dental for absorbing saliva and other fluids to help keep the area free of moisture.



Cautions:

1. Please check if the packaging is completed before use.
2. It's for single use only.

Cotton Pellets - Instruction for use:

- **Remove Cotton Pellets:** Take the desired number of cotton pellets from the bag. Ensure to handle them with clean hands to maintain hygiene.
- **Application:** Use the cotton pellets for your intended purpose, whether it's to dry cavities or absorb saliva during dental procedure while placing the cotton pellets in the patient's mouth.
- **Removal:** Carefully remove the cotton pellets from the patient's mouth after the procedure is completed. Ensure that no cotton fibers are left behind.
- **Disposal:** Dispose of used cotton pellets in accordance with your dental practice's waste disposal protocols. Do not reuse cotton pellets to prevent contamination.

Storage: Store in the well ventilated rooms, temperature control $\leq 40^{\circ}\text{C}$, moisture $\leq 80^{\circ}\text{C}$. Keep away from acid and avoid direct sunlight during storage and transportation.

Shelf life: 5 years



 00966553469922

 info@sif-manufacturing.com

 BOX 88992-RIYADH 11692

 sif-manufacturing.com



Sif Medical

Manufacturing

Labels, Packing Logo Design:

Symbol	Introductions	Symbol	Introductions
	Lot Number / Batch Code		Do not reuse" are "single use, "Use only once
	Warnings and Precautions		non-sterile
	medical device		Manufacture Date
	Manufacturer Name & Address		Expiry Date
	Do not use if package is damage		Consult Instructions for use
	No direct sunlight		Keep dry
	Upper limit of temperature		Upper limit of humidity





Sif Medical Manufacturing

Manufacturer Information



Saudi Industrialists Factory for Plastic Products
Factory No.105, Zone A,
Sudair City for Industry & Business, Riyadh - KSA
Tel: +966 112774111 / +966 553469922
Website: www.sif-manufacturing.com E-mail: info@sif-manufacturing.com

Difference between models :

The main difference between the below models are its size and its use for absorbing the fluids based on the type of dental procedure.

Cotton Pellet Models :

Cotton Pellets CP 001 – 11/32" (9mm)

Cotton Pellets CP 002 – 9/32" (7mm)

Cotton Pellets CP 003 – 7/32" (5.5mm)

Cotton Pellets CP 004 – 5/32" (4mm)

Cotton Pellets CP 005 – 1/8" (3mm)

Cotton Pellets CP 006 – 3/32" (2.4mm)

The difference between the cotton pellets are its size of the pellet ranges from larger size of 9mm to smaller size of 2.4mm as mentioned above.

Note : Occurrence of any serious incidents should report to the manufacturer and SFDA (19999) immediately.



00966553469922

info@sif-manufacturing.com

BOX 88992-RIYADH 11692

sif-manufacturing.com