



Sif Medical
Manufacturing

Instructions for Use

Name: Cotton Filled Gauze Sponges

Intend Use: The Cotton filled Gauze Sponges is intended to be used in dental for absorbing saliva and other fluids to help keep the area free of moisture.



Cautions:

1. Please check if the packaging is completed before use.
2. It's for single use only.

Cotton Gauze - Instruction for use:

- **Preparation:** Ensure hands are clean and sanitized before handling the gauze pads.
- **Selecting the Gauze:** Choose an appropriate number of gauze pads based on the procedure and the amount of absorbency needed.
- **Placement:** Position the sponge in the patient's mouth to absorb excess moisture and saliva.
- **Application:** Use the sponge to gently dab or press against the area as needed, ensuring it effectively absorbs fluids.
- **Monitoring:** Replace the sponge if it becomes saturated to maintain a clean working environment and optimal visibility during the procedure.
- **Disposal:** Dispose of used cotton gauze in accordance with your dental practice's waste disposal protocols. Do not reuse cotton pellets to prevent contamination.

Storage: Store in the well ventilated rooms, temperature control $\leq 40^{\circ}\text{C}$, moisture $\leq 80^{\circ}\text{C}$. Keep away from acid and avoid direct sunlight during storage and transportation.

Shelf life: 5 years



 00966553469922

 info@sif-manufacturing.com

 BOX 88992-RIYADH 11692

 sif-manufacturing.com



Sif Medical Manufacturing

Labels, Packing Logo Design:

Symbol	Introductions	Symbol	Introductions
	Lot Number / Batch Code		Do not reuse" are "single use, "Use only once
	Warnings and Precautions		non-sterile
	medical device		Manufacture Date
	Manufacturer Name & Address		Expiry Date
	Do not use if package is damage		Consult Instructions for use
	No direct sunlight		Keep dry
	Upper limit of temperature		Upper limit of humidity



00966553469922

info@sif-manufacturing.com

BOX 88992-RIYADH 11692

sif-manufacturing.com



Sif Medical Manufacturing

Manufacturer Information



Saudi Industrialists Factory for Plastic Products
Factory No.105, Zone A,
Sudair City for Industry & Business, Riyadh - KSA
Tel: +966 112774111 / +966 553469922
Website: www.sif-manufacturing.com E-mail: info@sif-manufacturing.com

Cotton filled Gauze Sponges Model :

CG 001 – 2" x 2", 8 ply

Note : Occurrence of any serious incidents should report to the manufacturer and SFDA (19999) immediately.



 00966553469922

 info@sif-manufacturing.com

 BOX 88992-RIYADH 11692

 sif-manufacturing.com