



Instructions for Use

Name: Air Water Syringe Tip (AWT MC)

Intend Use: Air-water syringe tip is used in dentistry to deliver a stream of air or water to the oral cavity.



Cautions:

1. Please check if the packaging is completed before use.
2. It's for single use only.

Instruction for use:

1. Take out the Air/water syringe tip from package, insert the end with card slot into Air/Water Gun. Straight Air/Water Syringe Tips are used for Air Water Gun with converter, just insert the Air/Water Syringe Tip into converter.
2. After installation, pull it lightly to check whether it's properly and firmly installed.
3. To make Air/Water Syringe Tips spout water, gas or mist, you need to press the Water and Gas button separately or simultaneously.
4. Dispose after single use.

Storage: Store in the well ventilated rooms, temperature control $\leq 40^{\circ}\text{C}$, moisture $\leq 80^{\circ}\text{C}$. Keep away from acid and avoid direct sunlight during storage and transportation.

Labels, Packing Logo Design:

Symbol	Introductions	Symbol	Introductions
	Lot Number / Batch Code		Do not reuse" are "single use, "Use only once
	Warnings and Precautions		non-sterile
	medical device		Manufacture Date
	Manufacturer Name & Address		Expiry Date





Sif Medical Manufacturing

	No direct sunlight		Keep dry
	Upper limit of temperature		Upper limit of humidity

Manufacturer Information



Saudi Industrialists Factory for Plastic Products
Factory No.105, Zone A,
Sudair City for Industry & Business, Riyadh - KSA
Tel: +966 112774111 / +966 553469922
Website: www.sif-manufacturing.com E-mail: info@sif-manufacturing.com

Note : Occurrence of any serious incidents should report to the manufacturer and SFDA (19999) immediately.



 00966553469922

 info@sif-manufacturing.com

 BOX 88992-RIYADH 11692

 sif-manufacturing.com