



Instructions for Use

Name: Barrier Film

- 1) SIF Barrier Film BF 001 – 4" x 6" Clear, 1200 sheets per roll
- 2) SIF Barrier Film BF 002 – 4" x 6" Blue, 1200 sheets per roll

Intend Use: Barrier Films are intended to be used to prevent cross contamination by providing a physical barrier between instruments, surfaces, clinicians and patients.



Cautions:

- 1. Please check if the packaging is completed before use.
- 2. It's for single use only.

Instruction for use:

- 1. Select the appropriate specification and take out the barrier film / sleeves;
- 2. Put the surgical equipment into the barrier sleeves, or apply the barrier film to the surface of area to be protected.
- 3. Dispose after use.

Storage: Store in the well ventilated rooms, temperature control $\leq 40^{\circ}$ C, moisture $\leq 80^{\circ}$. Keep away from acid and avoid direct sunlight during storage and transportation.

Shelf life: 5 years

Labels, Packing Logo Design:

Symbol	Introductions	Symbol	Introductions
	Lot Number / Batch Code		Do not reuse" are "single use, "Use only once
	Warnings and Precautions		non-sterile
	medical device		Manufacture Date
	Manufacturer Name & Address		Expiry Date





Sif Medical Manufacturing

	Do not use if package is damage		Consult Instructions for use
	No direct sunlight		Keep dry
	Upper limit of temperature		Upper limit of humidity

Manufacturer Information



Saudi Industrialists Factory for Plastic Products
Factory No.105, Zone A,
Sudair City for Industry & Business, Riyadh - KSA
Tel: +966 112774111 / +966 553469922
Website: www.sif-manufacturing.com E-mail: info@sif-manufacturing.com

Note : Occurrence of any serious incidents should report to the manufacturer and SFDA (19999) immediately



00966553469922

info@sif-manufacturing.com

BOX 88992-RIYADH 11692

sif-manufacturing.com