



Instructions for Use

Name: Dental Impression Trays

The variants are as follows,

- 1) Perforated Impression Tray (Large) – PIT 001
 - 2) Perforated Impression Tray (Medium) – PIT 002
 - 3) Perforated Impression Tray (Small) – PIT 003
 - 4) Fluoride Impression Tray (Dual Arch Large) – FIT 001
 - 5) Fluoride Impression Tray (Dual Arch Medium) – FIT 002
 - 6) Fluoride Impression Tray (Dual Arch Small) – FIT 003
 - 7) Fluoride Impression Tray (Single Arch Large) – FIT 004
 - 8) Fluoride Impression Tray (Single Arch Medium) – FIT 005
 - 9) Fluoride Impression Tray (Single Arch Small) – FIT 006
 - 10) Bite Registration Tray Anterior – BTA
 - 11) Bite Registration Tray Posterior – BTP
 - 12) Bite Registration Tray Quadrant – BTQ
 - 13) Bite Registration Tray Sideless – BTS
- Bite Registration Tray Full Arch – BTF

Intend Use: The device is used for oral impression for duplication of teeth.



Cautions:

1. Please check if the packaging is completed before use.
2. The device should be used before expiry date.
3. It's for single use only. Reuse may cause cross-contamination.

Instruction for use:

- 1 Determine and select the appropriate size and type of impression tray for the patient.
- 2 Fill the selected tray with the impression material, ensuring not to extend the material beyond the back of the impression tray.
- 3 Immediately place the loaded impression tray in the patient's mouth, patient bite the tray to make full teeth into the groove so as to make impression.
- 4 Once the impression is complete, remove the tray from the patient's mouth.
- 5 Dispose the tray after single use.

Storage: The product should be stored in dry place and keep away from acid, heat and direct sunlight during storage and transportation.





Sif Medical Manufacturing

Shelf life: 5 years

Labels, Packing Logo Design:

Symbol	Introductions	Symbol	Introductions
	Lot Number / Batch Code		Do not reuse" are "single use, "Use only once
	Warnings and Precautions		non-sterile
	medical device		Manufacture Date
	Manufacturer Name & Address		Expiry Date
	Mo direct sunlight		Keep dry

Manufacturer Information



Saudi Industrialists Factory for Plastic Products
Factory No.105, Zone A,
Sudair City for Industry & Business, Riyadh - KSA
Tel: +966 112774111 / +966 553469922
Website: www.sif-manufacturing.com E-mail: info@sif-manufacturing.com

Note : Occurrence of any serious incidents should report to the manufacturer and SFDA (19999) immediately.



 00966553469922

 info@sif-manufacturing.com

 BOX 88992-RIYADH 11692

 sif-manufacturing.com